



FRAME BARN

where great eyedeas are found
www.frame-barn.com 416.897.8779

presents

FWTNS

FRAME WORX - TITANIUM NOSE PAD SYSTEM Step by Step Instructions



1. Cut original nosepad/plastic pad arm using Frame Barn's FM706 Side Cutter.



2. Use a rough file to remove any large pieces left behind, then use a fine file to smooth the surface.



3. Use a marker to locate 2 drill points, then use a 1.0mm drill bit to drill 2 holes on each side. Each 2.00mm deep.



4. Use Frame Barn's FM704 Padded Metal Endpiece Angling Plier to insert the titanium pad arms into the 2 pre-drilled holes.



5. Insert the silicone nose pads (available in clear or black) onto the titanium pad arms.



6. Use Frame Barn's FM707 Pad Arm Adjusting Plier to adjust the nose pad as required.